



Item number | 1999915798



Briefs, ABENA Slip, XXL2, purple colourcode, Premium

- ✓ Breathable backsheet foil for healthy skin balance
- ✓ Top-Dry Technology for dry surface to protect the skin
- ✓ Flexible side panels for easy and optimum fit
- ✓ Leakage barriers for extra protection
- ✓ Wetness indicator
- ✓ Odor Control



Product Description

ABENA Slip Bariatric is our bladder protection briefs for moderate to heavy incontinence and designed for people with hip-size up to 254 cm. The flexible side panels with velcro tapes ensure an easy and optimum fit. The products are fully breathable and feature the Top-Dry Technology for optimum comfort ensuring complete skin respiration. They are also fitted with standing leakage barriers for extra protection. The Odor control system minimizes unpleasant smells. The ink print functioning as wetness indicator guides the user and caretaker to the product with the adequate absorption level and aids product changes.





















Specifications

| Base name | Briefs | | |
|--|---|--|--|
| Brand | ABENA | | |
| Sub-brand | Slip | | |
| Size | XXL2 | | |
| Color | White | | |
| Features | Purple colourcode | | |
| Material | ECF fluff, elastane, nonwoven, PE, PP, resin, SAP, textile-like backsheet (TBS) | | |
| Length/depth | 1030 mm | | |
| Width | 750 mm | | |
| Hip size | 170-254 | | |
| Tolerance | +/- 10% | | |
| Absorption Rothwell | 3200 ml | | |
| Gender | Unisex | | |
| Quality level | Premium | | |
| Certifications | CE, MD, UKCA | | |
| CE Class (Medical Devices) | Class I | | |
| Product or test standards | ISO 11948-1, ISO 13485, ISO 14001, ISO 9001 | | |
| Directives, regulations and acts | MDR (EU) 2017/745 | | |
| Safety Instructions and Warnings | Keep out of reach of children | | |
| Because of the products composition it has an expiry date from the production date on: | 5 years | | |
| Storage Instructions | Store dry, room temperature and no direct sunlight. | | |
| Product Disposal Instructions | Can be disposed of with normal household waste sorted according to local regulations. | | |



Packaging Disposal Instructions

To be recycled or incinerated.

Instructions for use/application

Bladder protection briefs for moderate to heavy incontinence. It is important to always measure the hip circumference to select the right size and to choose the correct absorption level to avoid leakages.

Fold the brief lengthways with the backsheet turned outwards to create a bowl shape. Pass the brief from front to back. Fan out the front of the brief, creating a channel shape between the legs. Fix the velcro tapes on both sides for a secure and comfortable fit. Easy product size recognition by color codes, names and numbering systems.

Packaging data

| Unit | Contains | Length | Width | Height | EAN |
|------|----------|---------|---------|---------|---------------|
| cll | 40 pcs | 44 cm | 37.9 cm | 30.4 cm | 5703538486036 |
| pck | 10 pcs | 22.9 cm | 18.2 cm | 28.8 cm | 5703538486029 |
| pcs | 1 pcs | | | | |



Regulation (EU) 2017/745 is a regulation of the European Union on the clinical investigation and sale of medical devices for human use. It repeals Directive 93/42/EEC, which concerns medical devices, and Directive 90/385/EEC, which concerns active implantable medical devices, on 26 May 2021.



The CE mark guarantees that a product is safe to use and complies with all safety precautions. CE stands for Conformité Européenne (European Conformity) and is mainly found on electronic equipment, safety equipment, construction products, and medical devices.



The UKCA mark guarantees that a product is safe to use and complies with all safety precautions in line with product standards and legislation of the UK.



The product is fully breathable for the comfort of the user; also during long-time use. Helps maintain skin integrity.



Flexible side panels with added elastics for easy application. A snug and secure fit supporting all body shapes.



Odor control is a critical factor in ensuring discreteness for the user. ABENA's incontinence products are designed and engineered to ensure that minimal Odor arises from the product.



Effective leakage barriers, both visible and invisible and latex-free elastics ensure a snug fit, which reduces the risk of leakages.



Unique Top-Dry Technology that provides a rapid absorption and ensures a dry surface even after multiple voids.



Wetness indicator is a liquidsensitive strip that changes colour or fades out indicating when it is time to change the product.



A combination of 100% breathability, Top-Dry and leakage protection provides a dry surface and healthy skin even after multiple voids maintaining the skin's natural pH balance.



The product does not contain



Drop system indicating absorption. The more drops, the higher the absorption. Ranges from 1 to 9 drops.





The bag is 30% PCR, which means made from plastic materials used by consumers. This will allow us to reduce our plastic waste and to help take care of the environment.





