

Sanisoft mini kol

with wings

For small anal leakage

sanicare 
your daily life



Sanisoft mini kol- is a unique protection pad for small anal leakage. This is a disposable protection pad with wings. It is thin, discrete and odour-inhibiting, with an effective carbon filter.





Tests performed by RISE (Research Institutes of Sweden) indicate that the product eliminates odour for up to 5 hours

Brand	Sanisoft
Designation	Sanisoft mini kol
Sanicare art no.	SA3536
Number of products/packaging	25 pc
Number of products/transport packaging	250 pc
Theoretical abs.kap ISO 11948-1 ml	143 ml
Practical abs.kap ISO 11948-1/3 ml	47,6 ml
Absorption material	Air-laid av cellulosa, super-absorbenter
Outer layer	Non-woven of polypropylene
Barrier layer	Polyethylene
Carbon filter	Active carbon and viscose
Dimensions	W 7,5-15 x L 30 cm
Colof of the release strip	White with dark purple text
Disposal	Flammable material, do not flush down toilet.
Storage	Room temperature
Additional information	The product does not contain PVC, latex, fragrance or skin care products. Instructions/information for the product may be obtained upon request.

User instructions and advice for use:

- Attach the product in underpants or fixation pants with the adhesive strip. Carefully remove the protective strip and apply the pad as illustrated.
- Make sure to use pants with a close fit, and fasten the pad in place for optimal function and to reduce the risk of leakage.

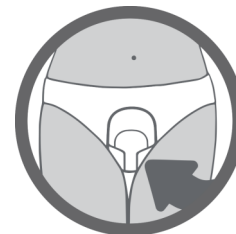

 For sore or infected skin, change the pad after each leakage. For heat rash: Air dry the skin. Change any wet pads, and change often. Ensure that correct size is used for optimal function. Do not place in direct sun, or near or in direct contact with objects and devices that emit fire or high heat. In the event of serious product defects or incidents, contact the manufacturer and the Swedish Medical Products Agency.

 Sanicare AB
Box 260
311 23 Falkenberg

 SWEDISH
Medtech



Art.nr.
SA3536



sanicare
your daily life

Sanicare AB, Box 260, 311 23 Falkenberg. Phone: +46 (0) 346-71 55 30, Fax: 0346-71 55 39
Email: sanicare@sanicare.se website: www.sanicare.se